



Quantification of Rare PD-L1 and Other Immunology Biomarker Expressing Leukocytes and CTCs in Peripheral Blood of Cancer Patients

Adam Jendrisak, Nadia Ebrahim, Angel Rodriguez, Rachel Krupa, David Lu, Mahipal Suraneni, Jiyun Byun, Ryon Graf, Yipeng Wang, Mark Landers, Ryan Dittamore
Epic Sciences, Inc., San Diego, California

Background

- Expression of PD-L1 on tumor tissue and immune cell markers (CD3, CD4, CD8, etc.) on tumor infiltrating lymphocytes (TILs) are often associated with improved response to PD-1 and PD-L1 checkpoint inhibitors. However, PD-L1 diagnostic tests suffer from high co-morbidities and from significant false positives and false negatives.
- Expression of PD-1 and Tim-3 on TILs is associated with immune exhaustion.
- Utilizing a non-invasive liquid biopsy, we sought to examine the expression of checkpoint markers (PD-L1, Tim-3) and immune cell markers (CD3, CD8) on circulating tumor cells (CTCs) and leukocyte cell populations for the purpose of developing an improved predictive and pharmacodynamic biomarker for approved PD-1/PD-L1 checkpoint inhibitors and novel immunotherapy drugs in development.


Methods

- Blood samples were drawn from 3 healthy donors, 18 non-cancerous lung disease, 33 lung cancer, and 1 bladder cancer patients and sent to Epic Sciences for processing with Epic Sciences' immunoassays.
- Panels of immunoassays include staining of checkpoint markers (PD-L1, Tim-3) and immune cell markers (CD3, CD8).

1) SLIDE PREPARATION



2) CELL STAINING

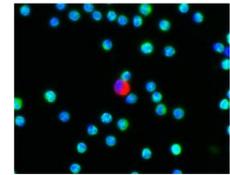

CK, CD45, DAPI, PD-L1, etc.


3) SCANNING



4) SINGLE CELL DIGITAL PATHOLOGY




5) BIOMARKER ANALYSIS ON CTC AND LEUKOCYTES

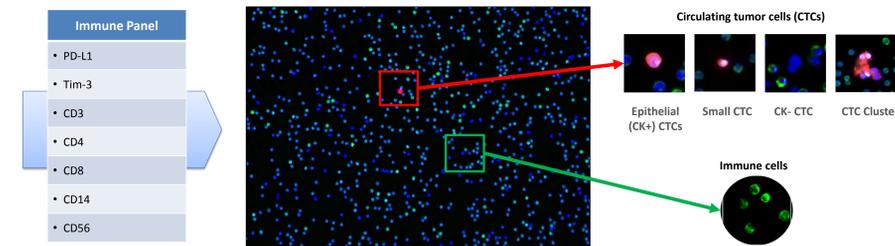


Schematic of Epic CTC platform CTC enumeration, morphology and protein analyses workflow:

- 1) Nucleated cells from one 10mL blood draw were plated onto 10-12 slides with each slide having ~3 million cells
- 2) Slides stained with DAPI and a combination of two or three other markers, including cytokeratin (CK), CD3, CD8, CD45, PD-L1 and Tim-3
- 3) Slides scanned
- 4) All cells measured by a multi-parametric digital pathology algorithm for morphological features and protein expression
- 5) Biomarker analyses were performed on both CTCs and leukocytes


PD-L1 Protein Assay Development

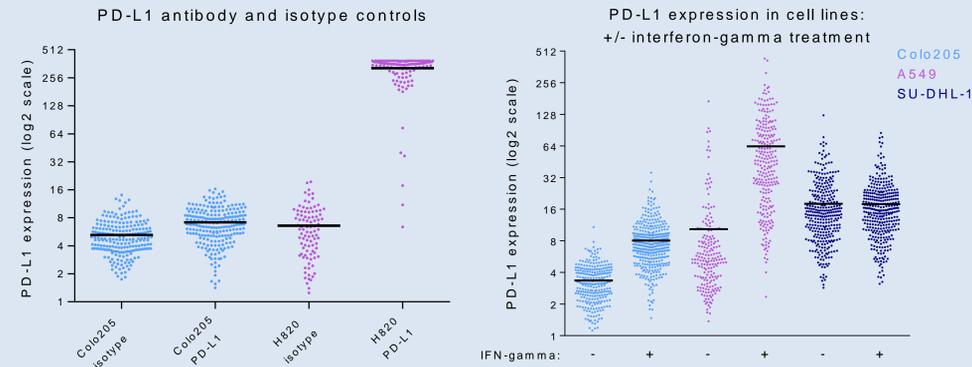

PD-L1 Expression in CTC and Leukocytes of Lung Ca

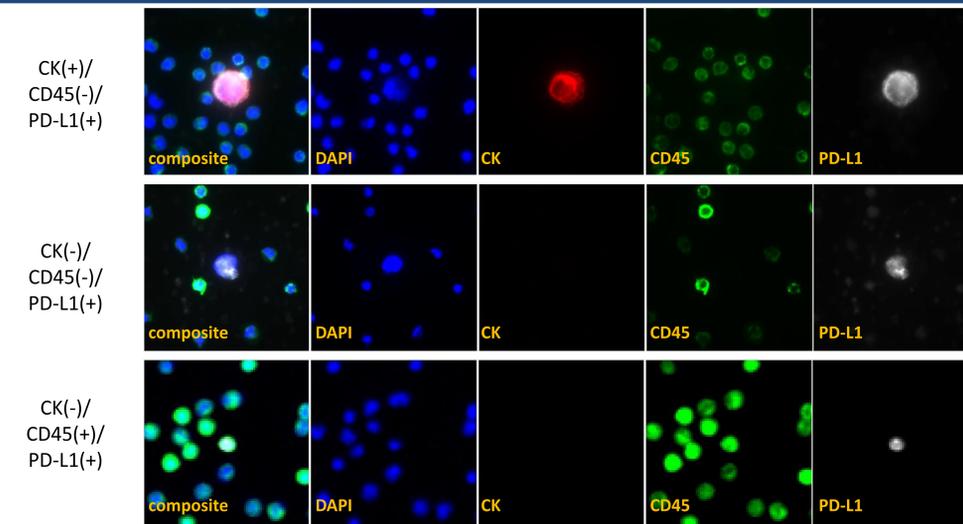


Percentage of PD-L1(+) Leukocytes in 54 Sample Cohort

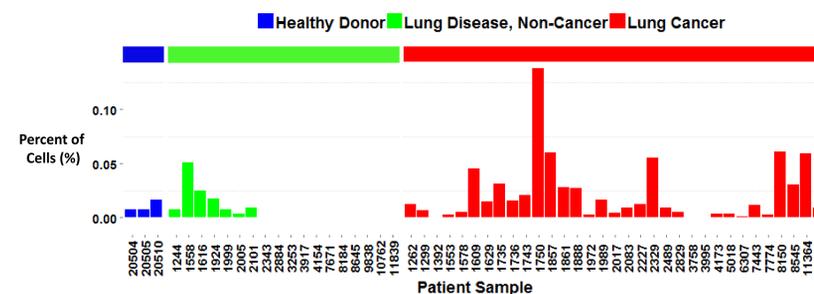


	Minimum	Maximum	Median	Mean
Healthy Donor	0.0076%	0.0166%	0.0077%	0.0106%
Lung Disease (non-Cancerous)	0.0000%	0.0510%	0.0000%	0.0067%
Lung Cancer	0.0000%	0.1378%	0.0118%	0.0213%


Tim-3 Expression in Healthy Donor Leukocytes

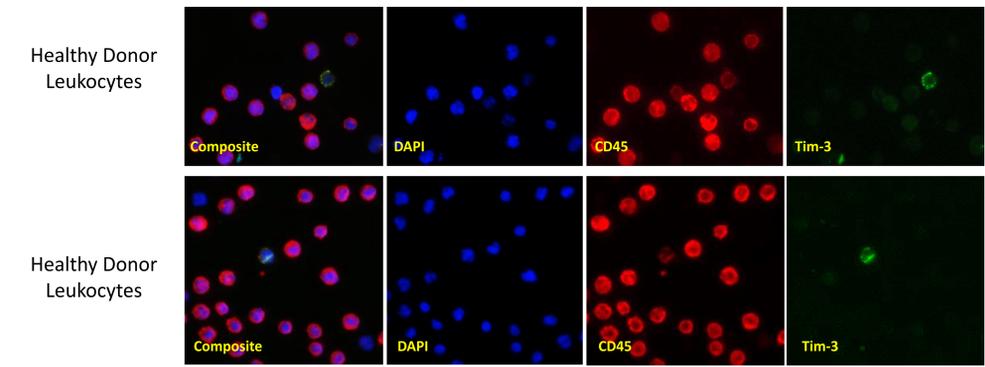

CD3 Expression in Jurkat Cells and Healthy Donor Leukocytes

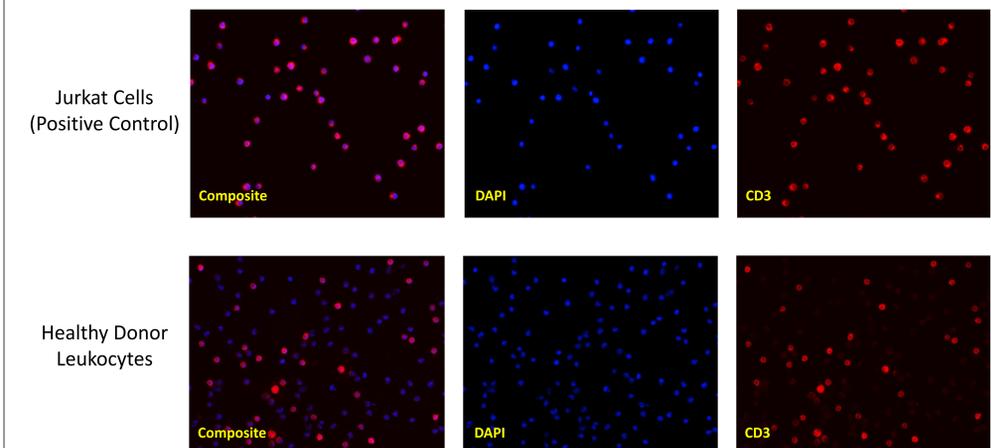

CD8, CD45 and PD-L1 Co-Expression in Bladder Ca Leukocytes

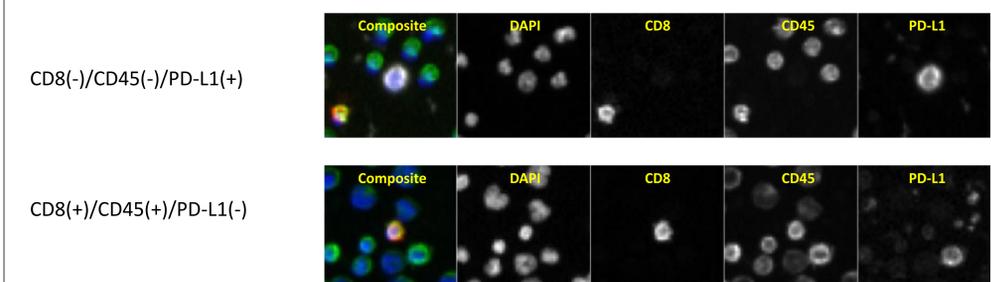

Conclusions

- Epic Sciences' platform has low limit of detection, ability to archive patient blood samples and ability to quantify biomarker expression on both CTCs and leukocytes simultaneously.
- Detection of leukocyte subtypes such as CD3, CD8, CD14 and CD56 cells, will allow us to further characterize PD-L1 and Tim-3 in T-lymphocytes and other immune cell types.
- Development of a liquid biopsy-based platform that is capable of simultaneously measuring immune biomarkers in CTCs as well as leukocytes will allow real time assessment of response to immune checkpoints inhibitors.